**Sterile Cannulated Drills and Headed Reamers**  
**Fresadoras canuladas estériles y taladradoras con cabeza**  
**Forets perforés stériles et alésoirs stériles à tête**  
**Sterile kanülierte Bohrer und Kopfreibahnen**  
**Punte cannulate e alesatori con testa sterili**  
**Brocas Canuladas e Escareadores com Cabeça Esterilizados**

**Important Product Information**  
**Información importante sobre el producto**  
**Informations importantes relatives au produit**  
**Wichtige Produktinformationen**  
**Informazioni importanti sul prodotto**  
**Informações Importantes sobre o Produto**

**Instructions for Use**  
**Instrucciones de uso**  
**Mode d'emploi**  
**Gebrauchsanweisung**  
**Istruzioni per l'uso**  
**Instruções de uso**

31607 Rev 2  
Eff. 09/24

 Parcus Medical, LLC  
6423 Parkland Dr.  
Sarasota, FL 34243 USA

 **ANIKA**  
[www.anika.com](http://www.anika.com)  
(941)755-7965

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## **Sterile Cannulated Drills and Headed Reamers (EN)**

### **1. Intended Use:**

Cannulated Drills and Headed Reamers are used to create tunnels and sockets in bone. This product is designed to be used in conjunction with a Guide Pin or Guide Wire.

### **2. Warnings:**

- A. Sterile devices designed for single use only should not be re-sterilized.
- B. Reuse of a device that is intended for single use could result in inferior device performance or patient infection.
- C. Never use a bent, dull, or damaged drill bit/reamer.
- D. Never use a device that has discoloration or staining.
- E. Never use a device with faded laser mark(s).
- F. Never use a damaged drill guide, guide pin or guide wire.
- G. These devices are not intended for use as an implant or to be left in the body.
- H. Applying excess pressure or using the device in an off-axis manner may result in breakage/bending or damage to both drill/reamer and guide pin/guide wire.
- I. Limit duration of use. Excessive use without irrigation may lead to heat buildup at the implant site.

### **3. Precautions**

- A. Federal law restricts these devices to sale by or on the order of a physician.
- B. Stop using immediately if excessive force is required to advance the drill bit/reamer.
- C. Never use a device with any suspected defect or damage not covered in this IFU. If such damage is detected, notify the manufacturer.
- D. Common risks consistent with procedures using this type of device may include bone/tissue damage, intervention/ revision surgery, soft tissue irritation, loss of fixation/repair, infection, allergic reaction(s).

### **4. Material:**

These devices are manufactured from a Stainless-Steel alloy. The material used in the manufacture of the devices is radio-opaque and can, therefore, be detected with conventional X-Ray or fluoroscopy.

### **5. Inspection:**

- A. Inspect the device for damage at all stages of handling.
- B. If damage is detected, consult the manufacturer for guidance.

### **6. Packaging and Labeling:**

- A. Do not use this device if the packaging or labeling has been damaged, shows signs of exposure to moisture or extreme temperature or has been altered in any way.
- B. Contact the manufacturer to report any package/label damage or alterations.

### **7. Sterilization:**

These devices are sterilized by EO gas and supplied in sterile packaging. This product must never be re-sterilized.

### **8. Storage**

Devices must be stored in the original unopened package in a dry place and must not be used beyond the expiration date indicated on the package.

### **9. Instructions for Use:**

- A. Use an appropriately designed drill or wire driver to place a guide pin in the desired position.
- B. Open the product using aseptic technique.
- C. Securely attach the drill/reamer into the drill chuck.

- D. Place the drill/reamer over the guide pin.
- E. Make sure that the drill motor is set to rotate in a clockwise direction.
- F. Prudently advance the drill/reamer over the guide pin, ensuring to maintain a coaxial orientation between the pin and the drill/reamer.
- G. After the desired depth has been reached, reverse the drill, and remove the drill/reamer from the bone.

## 10. Device Disposal

Sterile devices that have been removed from the packaging and reusable devices with visible damage(s) must be properly disposed of in accordance with the institution's policy.

## 11. Adverse / Reportable Events

Any serious incident that occurs in relation to the device should be reported to both Anika Therapeutics and the applicable authority using the following contact information.

Country	Authority	Email	Website
All events	Anika Therapeutics	Globalcomplaints@anika.com	www.anika.com
Australia	Therapeutic Goods Administration (TGA)	IRIS@health.gov.au	www.tga.gov.au/reporting-adverse-events
European Union	EC Representative	info@mdssar.com	www.mdssar.com

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